NEOMED[®]

- EN Instructions for Use
- FR Mode d'emploi
- ES Instrucciones de uso
- PT Instruções de utilização
- DK Brugsanvisning
- NL Gebruiksaanwijzing
- DE Gebrauchsanweisung
- NO Bruksanvisning
- SV Bruksanvisning

- EN Enteral Extension Set with Enteral Only Connector
- FR Ensemble de rallonge entérale doté d'un connecteur entéral uniquement
- ES Juego de extensión gástrica con conector gástrico
- PT Conjunto de extensão entérica com conector apenas para via entérica
- DK Forlængersæt med konnektor udelukkende til enteral anvendelse
- NL Enterale verlengingsset met Uitsluitend enteraal-aansluiting
- DE Enteraler Verlängerungssatz mit NUR-ENTERAL-Anschluss
- NO Forlengersett for enteral ernæring, med kobling kun for enteral bruk
- SV Förlängningssats för enteral bruk med enbart enteral anslutning

NeoMed Incorporated
100 Londonderry Ct. Suite 112
Woodstock, GA 30188 USA
(770) 516-2225
www.neomedinc.com

EC REP

Emergo Europe,
Prinsessegracht 20,
2514 AP, The Hague,
The Netherlands

Australian Sponsor: Emergo Australia
Level 20, Tower II Darling Park
201 Sussex Street, Sydney, NSW 2000
Australia

ENGLISH

This instructions for use applies to all NeoMed Extension Sets.

INDICATIONS FOR USE:

The NeoMed Enteral Only Extension Set is indicated for use in neonatal and pediatric patients in connection with a nasogastric/orogastric enteral feeding tubes to provide nutrition via nasal or oral gastric placements.

CONTRAINDICATIONS:

This device is not intended for I.V. use.

FEATURES:

The NeoMed Enteral Only Extension Set incorporates safety connectors which help mitigate the risk of accidental connection of an I.V. system to the enteral system or the enteral system to an I.V. system. NeoMed Enteral Only Extension sets are not made with natural rubber latex, DEHP or BPA.

NeoMed extension sets are provided in a variety of configurations:

- Standard extension sets have one proximal and one distal enteral port. Enteral ports can be male or female.
- Bifurcated extension sets allow for medication or additional nutritional delivery without disrupting the feeding line.
- Bifurcated extension sets provide an additional proximal enteral port.
- Double bifurcated extension sets provide two proximal enteral ports and two distal enteral ports.
- Large bore extension sets facilitate delivery of thicker (more viscous) nutrition.
- Extension sets with a drip chamber provide a visual indicator of nutritional flow.
- Sampling Straws allow for aseptic withdrawal and transfer of fluid from a container, and are not designed to directly connect to an enteral feeding tube.

CAUTIONS:

1. Do not wash and reuse extension set. It is intended for one time use only. Re-use of this device may increase risk of leakage or biological infection.
2. When using bifurcated extension sets, always close slide clamp and plug female hub of bifurcated tubing when not in use.

DIRECTIONS:

1. Always follow physician's instructions for use of contents.
2. Wash hands thoroughly and use aseptic technique throughout process.
3. Remove extension set from packaging.
4. Verify feeding tube compatibility prior to use.
5. Attach Enteral Only Connector (inlet) to outlet of administration set. Attach extension set outlet to enteral only feeding tube.
6. Close slide clamp and plug all female hub's until ready to use or when not in use.
7. Replace and maintain administration set in accordance to hospital protocol and not to exceed NANN or ADA guidelines of beyond 4 hours, as applicable.
8. Do not re-use extension sets. After use, dispose of device per hospital protocols and/or local or national regulations.

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